

SIEMENS MAGNETOM Investigational_Device_7T syngo MR B17

\\USER\SHB\WPC-8922\SPA_31P\anat-scout_ses-01p31

TA: 0:13 PAT: Off Voxel size: 1.1x1.0x7.0 mm Rel. SNR: 1.00 SIEMENS: gre

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	On
Start measurements	single

Routine

Slice group 1	
Slices	1
Dist. factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Slice group 2	
Slices	1
Dist. factor	20 %
Position	L0.0 A12.4 F6.6
Orientation	Transversal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Slice group 3	
Slices	1
Dist. factor	20 %
Position	L0.0 A14.8 F24.5
Orientation	Coronal
Phase enc. dir.	R >> L
Rotation	0.00 deg
Phase oversampling	0 %
FoV read	250 mm
FoV phase	100.0 %
Slice thickness	7.0 mm
TR	8.6 ms
TE	4.00 ms
Averages	2
Concatenations	3
Filter	Normalize, Elliptical filter
Coil elements	1H

Contrast

TD	0 ms
MTC	Off
Magn. preparation	None
Flip angle	20 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution

Base resolution	256
-----------------	-----

Phase resolution	90 %
Phase partial Fourier	Off
Interpolation	On

PAT mode	None

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	Off
Normalize	On
Intensity	Medium
Cut off	20
Width	4
Unfiltered images	Off
B1 filter	Off
Raw filter	Off
Elliptical filter	On
Mode	Inplane

Geometry

Multi-slice mode	Sequential
Series	Interleaved

Saturation mode	Standard
Special sat.	None

Table position	H
Table position	0 mm
Inline Composing	Off

Tim CT mode	Off

System

1H	On
31P	Off

Positioning mode	REF
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Save uncombined	On
Coil Combine Mode	Sum of Squares
AutoAlign	---
Auto Coil Select	Default

Shim mode	Tune up
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
! Ref. amplitude 1H	200.000 V
Adjustment Tolerance	Auto
Adjust volume	
! Position	L1.2 A10.8 F21.5
! Orientation	C > T39.5
! Rotation	90.00 deg
! R >> L	192 mm
! F >> H	167 mm
! A >> P	191 mm

Physio

1st Signal/Mode	None
Segments	1

Tagging	None
Dark blood	Off

Resp. control	Off

Inline

Subtract	Off
Liver registration	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off

MapIt	None
Contrasts	1

Sequence

Introduction	On
Dimension	2D
Phase stabilisation	Off
Asymmetric echo	Allowed
Bandwidth	320 Hz/Px
Flow comp.	No

RF pulse type	Normal
Gradient mode	Normal
Excitation	Slice-sel.
RF spoiling	On

SIEMENS MAGNETOM Investigational_Device_7T syngo MR B17

\\USER\SHB\WPC-8922\SPA_31P\anat-scout_ses-01p31

TA: 0:13 PAT: Off Voxel size: 1.1x1.0x7.0 mm Rel. SNR: 1.00 SIEMENS: gre

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	On
Start measurements	single

Routine

Slice group 1	
Slices	1
Dist. factor	20 %
Position	R1.8 A50.0 F13.4
Orientation	Sagittal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Slice group 2	
Slices	1
Dist. factor	20 %
Position	R1.8 A62.4 F20.0
Orientation	Transversal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Slice group 3	
Slices	1
Dist. factor	20 %
Position	R1.8 A64.9 F37.9
Orientation	Coronal
Phase enc. dir.	R >> L
Rotation	0.00 deg
Phase oversampling	0 %
FoV read	250 mm
FoV phase	100.0 %
Slice thickness	7.0 mm
TR	8.6 ms
TE	4.00 ms
Averages	2
Concatenations	3
Filter	Normalize, Elliptical filter
Coil elements	1H

Contrast

TD	0 ms
MTC	Off
Magn. preparation	None
Flip angle	20 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution

Base resolution	256
-----------------	-----

Phase resolution	90 %
Phase partial Fourier	Off
Interpolation	On

PAT mode	None

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	Off
Normalize	On
Intensity	Medium
Cut off	20
Width	4
Unfiltered images	Off
B1 filter	Off
Raw filter	Off
Elliptical filter	On
Mode	Inplane

Geometry

Multi-slice mode	Sequential
Series	Interleaved

Saturation mode	Standard
Special sat.	None

Table position	H
Table position	0 mm
Inline Composing	Off

Tim CT mode	Off

System

1H	On
31P	Off

Positioning mode	REF
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Save uncombined	On
Coil Combine Mode	Sum of Squares
AutoAlign	---
Auto Coil Select	Default

Shim mode	Tune up
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
! Ref. amplitude 1H	200.000 V
Adjustment Tolerance	Auto
Adjust volume	
! Position	R2.3 A48.3 F29.6
! Orientation	C > T39.5
! Rotation	90.00 deg
! R >> L	192 mm
! F >> H	167 mm
! A >> P	191 mm

Physio

1st Signal/Mode	None
Segments	1

Tagging	None
Dark blood	Off

Resp. control	Off

Inline

Subtract	Off
Liver registration	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off

MapIt	None
Contrasts	1

Sequence

Introduction	On
Dimension	2D
Phase stabilisation	Off
Asymmetric echo	Allowed
Bandwidth	320 Hz/Px
Flow comp.	No

RF pulse type	Normal
Gradient mode	Normal
Excitation	Slice-sel.
RF spoiling	On

SIEMENS MAGNETOM Investigational_Device_7T syngo MR B17

\\USER\SHB\WPC-8922\SPA_31P\anat-scout_ses-01p31

TA: 0:13 PAT: Off Voxel size: 1.7x1.6x7.0 mm Rel. SNR: 1.00 SIEMENS: gre

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	On
Start measurements	single

Routine

Slice group 1	
Slices	1
Dist. factor	20 %
Position	R1.8 A50.0 F13.4
Orientation	Sagittal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Slice group 2	
Slices	1
Dist. factor	20 %
Position	R1.8 A62.4 F20.0
Orientation	Transversal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Slice group 3	
Slices	1
Dist. factor	20 %
Position	R1.8 A64.9 F37.9
Orientation	Coronal
Phase enc. dir.	R >> L
Rotation	0.00 deg
Phase oversampling	0 %
FoV read	400 mm
FoV phase	100.0 %
Slice thickness	7.0 mm
TR	8.6 ms
TE	4.00 ms
Averages	2
Concatenations	3
Filter	Normalize, Elliptical filter
Coil elements	1H

Contrast

TD	0 ms
MTC	Off
Magn. preparation	None
Flip angle	20 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution

Base resolution	256
-----------------	-----

Phase resolution	90 %
Phase partial Fourier	Off
Interpolation	On

PAT mode	None

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	Off
Normalize	On
Intensity	Medium
Cut off	20
Width	4
Unfiltered images	Off
B1 filter	Off
Raw filter	Off
Elliptical filter	On
Mode	Inplane

Geometry

Multi-slice mode	Sequential
Series	Interleaved

Saturation mode	Standard
Special sat.	None

Table position	H
Table position	0 mm
Inline Composing	Off

Tim CT mode	Off

System

1H	On
31P	Off

Positioning mode	REF
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Save uncombined	On
Coil Combine Mode	Sum of Squares
AutoAlign	---
Auto Coil Select	Default

Shim mode	Tune up
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
! Ref. amplitude 1H	200.000 V
Adjustment Tolerance	Auto
Adjust volume	
! Position	R2.3 A48.3 F29.6
! Orientation	C > T39.5
! Rotation	90.00 deg
! R >> L	192 mm
! F >> H	167 mm
! A >> P	191 mm

Physio

1st Signal/Mode	None
Segments	1

Tagging	None
Dark blood	Off

Resp. control	Off

Inline

Subtract	Off
Liver registration	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off

MapIt	None
Contrasts	1

Sequence

Introduction	On
Dimension	2D
Phase stabilisation	Off
Asymmetric echo	Allowed
Bandwidth	320 Hz/Px
Flow comp.	No

RF pulse type	Normal
Gradient mode	Normal
Excitation	Slice-sel.
RF spoiling	On

SIEMENS MAGNETOM Investigational_Device_7T syngo MR B17

\\USER\SHB\WPC-8922\SPA_31P\fastestmap

TA: 0:12

Vol: 122 x169 x120 mm

Rel. SNR: 1.00

USER: fastestmap

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	single

Routine

Position	R1.5 A11.1 F11.9
Orientation	Transversal
Rotation	-90.00 deg
Vol R >> L	122 mm
Vol A >> P	169 mm
Vol F >> H	120 mm
TR	2000 ms
TE	49.20 ms
Averages	1
Filter	None
Coil elements	1H

Contrast

Tau	5.00 ms
Excite flip angle	90 deg
Refocus flip angle	180 deg
Measurements	1

Resolution

Vector size	256
-------------	-----

Geometry

Table position	H
Table position	0 mm
Inline Composing	Off

System

1H	On
31P	Off
Positioning mode	REF
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Save uncombined	Off
AutoAlign	---
Auto Coil Select	Default
Shim mode	Advanced
Adj. water suppr.	Off
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
! Ref. amplitude 1H	75.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	R1.5 A11.1 F11.9
Orientation	Transversal

Rotation	-90.00 deg
A >> P	169 mm
R >> L	122 mm
F >> H	120 mm

Physio

1st Signal/Mode	None
-----------------	------

Composing

Sequence

Delta frequency	0.0 ppm
Phase cycling	None
Bandwidth	100000 Hz
Acquisition duration	2 ms
Type of fit	Full 6-proj
Vol fit factor	100 %
Refocus pulses	Normal
Excite pulse duration	6400 us
Refocus pulse duration	6400 us
Bar FoV	384 mm
Bar thickness	10.0 mm
Multi-echo acquisition	On
Number of echoes	4
Inversion pulse	Off

SIEMENS MAGNETOM Investigational_Device_7T syngo MR B17

\\USER\SHB\WPC-8922\SPA_31P\anat-T1w_acq-raw

TA: 5:14 PAT: Off Voxel size: 1.2x1.2x1.2 mm Rel. SNR: 1.00 SIEMENS: tfl

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	On
Start measurements	single

Routine

Slab group 1	
Slabs	1
Dist. factor	50 %
Position	R3.1 A28.4 F24.2
Orientation	T > C-2.3 > S0.1
Phase enc. dir.	R >> L
Rotation	89.90 deg
Phase oversampling	0 %
Slice oversampling	33.3 %
Slices per slab	144
FoV read	231 mm
FoV phase	71.9 %
Slice thickness	1.20 mm
TR	3000 ms
TE	1.82 ms
Averages	1
Concatenations	1
Filter	B1 filter
Coil elements	1H

Contrast

Magn. preparation	Slice-sel. IR
T1	1200 ms
Flip angle	8 deg
Fat suppr.	None
Water suppr.	None
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Off

Resolution

Base resolution	192
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	6/8
Slice partial Fourier	6/8
Interpolation	Off
PAT mode	None
Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	Off
Normalize	Off
B1 filter	On
Intensity	Medium
Unfiltered images	On

Raw filter	Off
Elliptical filter	Off

Geometry

Multi-slice mode	Single shot
Series	Interleaved
Table position	H
Table position	0 mm
Inline Composing	Off

System

1H	On
31P	Off
Positioning mode	FIX
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Save uncombined	Off
Coil Combine Mode	Sum of Squares
AutoAlign	---
Auto Coil Select	Default
Shim mode	Standard
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
! Ref. amplitude 1H	150.000 V
Adjustment Tolerance	Auto
Adjust volume	
! Position	R1.5 A19.3 F6.4
! Orientation	Transversal
! Rotation	-90.00 deg
! A >> P	172 mm
! R >> L	140 mm
! F >> H	121 mm

Physio

1st Signal/Mode	None
Dark blood	Off
Resp. control	Off

Inline

Subtract	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Sequence

Introduction	On
Dimension	3D
Elliptical scanning	Off
Asymmetric echo	Allowed
Bandwidth	390 Hz/Px
Flow comp.	No
Echo spacing	4.5 ms
RF pulse type	Fast

SIEMENS MAGNETOM Investigational_Device_7T syngo MR B17

Gradient mode
Excitation
RF spoiling

Normal
Slab-sel.
On

SIEMENS MAGNETOM Investigational_Device_7T syngo MR B17

\\USER\SHB\WPC-8922\SPA_31P\mrs-mrsi_acq-31Pv160

TA: 0:40

Voxel size: 340.0x340.0x50.0 mm

Rel. SNR: 1.00

USER: csi_fid_dt1300

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	On
Start measurements	single

Shim mode	Standard
Adj. water suppr.	Off
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
! Ref. amplitude 31P	160.000 V
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
! Position	R1.5 A11.1 F11.9
! Orientation	Transversal
! Rotation	-90.00 deg
! A >> P	169 mm
! R >> L	122 mm
! F >> H	120 mm

Routine

Position	L0.2 A11.7 H3.2
Orientation	Transversal
Rotation	0 deg
FoV R >> L	340 mm
FoV A >> P	340 mm
Thickness F >> H	50 mm
Vol R >> L	340 mm
Vol A >> P	340 mm
Slices	1
TR	540 ms
TE	1.0 ms
Averages	64
Filter	Hamming
Coil elements	31P

Physio

1st Signal/Mode	None
-----------------	------

Composing

Sequence

Preparation scans	10
Dimension	2D
Delta frequency	0.0 ppm
Phase encoding	Weighted
Acquisition delay	0.8 ms
Bandwidth	8000 Hz
Acquisition duration	128 ms
Remove oversampling	On

Contrast

Averaging mode	Short term
Flip angle	90 deg
Water suppr.	None
Measurements	1

TX/RX Nucleus	31P
TX/RX delta frequency	0 Hz
NOE type	Rectangular
NOE count	2
NOE duration	5.0 ms
NOE pause	10.0 ms
NOE flip angle	90 deg
TX Nucleus	1H
TX delta frequency	0 Hz
Decoupling type	None

Resolution

Scan res. R >> L	1
Scan res. A >> P	1
Interpol. res. R >> L	1
Interpol. res. A >> P	1
Hamming	On
Width	100
Prescan Normalize	Off
Vector size	1024

Geometry

Table position	H
Table position	0 mm
Inline Composing	Off

System

1H	Off
31P	On
Positioning mode	FIX
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Save uncombined	Off
AutoAlign	---
Auto Coil Select	Default

SIEMENS MAGNETOM Investigational_Device_7T syngo MR B17

\\USER\SHB\WPC-8922\SPA_31P\mrs-mrsi_acq-31P

TA: 25:39

Voxel size: 24.3x24.3x24.3 mm

Rel. SNR: 1.00

USER: csi_fid_dt1300

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	On
Start measurements	single

Routine

Position	L0.2 A11.7 H3.2
Orientation	Transversal
Rotation	0 deg
FoV R >> L	340 mm
FoV A >> P	340 mm
FoV F >> H	340 mm
Vol R >> L	340 mm
Vol A >> P	340 mm
Vol F >> H	140 mm
Slabs	1
TR	540 ms
TE	1.0 ms
Averages	12
Filter	Hamming
Coil elements	31P

Contrast

Averaging mode	Short term
Flip angle	90 deg
Water suppr.	None
Measurements	1

Resolution

Scan res. R >> L	14
Scan res. A >> P	14
Scan res. F >> H	14
Interpol. res. R >> L	16
Interpol. res. A >> P	16
Interpol. res. F >> H	16
Hamming	On
Width	100
Prescan Normalize	Off
Vector size	1024

Geometry

Table position	H
Table position	0 mm
Inline Composing	Off

System

1H	Off
31P	On
Positioning mode	FIX
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H

Save uncombined	Off
AutoAlign	---
Auto Coil Select	Default

Shim mode	Standard
Adj. water suppr.	Off
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
! Ref. amplitude 31P	160.000 V
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
! Position	R1.5 A11.1 F11.9
! Orientation	Transversal
! Rotation	-90.00 deg
! A >> P	169 mm
! R >> L	122 mm
! F >> H	120 mm

Physio

1st Signal/Mode	None
-----------------	------

Composing

Sequence

Preparation scans	10
Dimension	3D
Delta frequency	0.0 ppm
Phase encoding	Weighted
Acquisition delay	0.8 ms
Bandwidth	8000 Hz
Acquisition duration	128 ms
Remove oversampling	On
TX/RX Nucleus	31P
TX/RX delta frequency	0 Hz
NOE type	Rectangular
NOE count	2
NOE duration	5.0 ms
NOE pause	10.0 ms
NOE flip angle	90 deg
TX Nucleus	1H
TX delta frequency	0 Hz
Decoupling type	None

SIEMENS MAGNETOM Investigational_Device_7T syngo MR B17

\\USER\SHB\WPC-8922\SPA_31P\anat-scout_acq-LD

TA: 1:56 PAT: Off Voxel size: 1.3x1.1x4.0 mm Rel. SNR: 1.00 SIEMENS: gre

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	single

Routine

Slice group 1	
Slices	85
Dist. factor	0 %
Position	L1.9 A9.9 F1.2
Orientation	Transversal
Phase enc. dir.	R >> L
Rotation	90.00 deg
Phase oversampling	0 %
FoV read	340 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
TR	449.0 ms
TE	2.46 ms
Averages	1
Concatenations	1
Filter	Elliptical filter
Coil elements	1H

Contrast

MTC	Off
Magn. preparation	None
Flip angle	47 deg
Fat suppr.	None
Water suppr.	None
SWI	Off
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution

Base resolution	320
Phase resolution	80 %
Phase partial Fourier	Off
Interpolation	Off
PAT mode	None
Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	Off
Normalize	Off
B1 filter	Off
Raw filter	Off
Elliptical filter	On
Mode	Inplane

Geometry

Multi-slice mode	Interleaved
------------------	-------------

Series Interleaved

Saturation mode	Standard
Special sat.	None

Table position	H
Table position	0 mm
Inline Composing	Off

Tim CT mode	Off
-------------	-----

System

1H	On
31P	Off

Positioning mode	FIX
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Save uncombined	Off
Coil Combine Mode	Adaptive Combine
AutoAlign	---
Auto Coil Select	Off

Shim mode	Standard
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
! Ref. amplitude 1H	100.000 V
Adjustment Tolerance	Auto
Adjust volume	
! Position	R1.5 A11.1 F11.9
! Orientation	Transversal
! Rotation	-90.00 deg
! A >> P	169 mm
! R >> L	122 mm
! F >> H	120 mm

Physio

1st Signal/Mode	None
Segments	1

Tagging	None
Dark blood	Off

Resp. control	Off
---------------	-----

Inline

Subtract	Off
Liver registration	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off

Maplt	None
-------	------

| Contrasts 1

Sequence

Introduction	On
Dimension	2D
Phase stabilisation	Off
Asymmetric echo	Allowed
Bandwidth	330 Hz/Px
Flow comp.	No

RF pulse type	Fast
Gradient mode	Fast*
Excitation	Slice-sel.
RF spoiling	On

SIEMENS MAGNETOM Investigational_Device_7T syngo MR B17

\\USER\SHB\WPC-8922\SPA_31P\anat-scout_acq-shift

TA: 0:13 PAT: Off Voxel size: 1.1x1.0x7.0 mm Rel. SNR: 1.00 SIEMENS: gre

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	On
Start measurements	single

Routine

Slice group 1	
Slices	1
Dist. factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Slice group 2	
Slices	1
Dist. factor	20 %
Position	L0.0 A12.4 F6.6
Orientation	Transversal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Slice group 3	
Slices	1
Dist. factor	20 %
Position	L0.0 A14.8 F24.5
Orientation	Coronal
Phase enc. dir.	R >> L
Rotation	0.00 deg
Phase oversampling	0 %
FoV read	250 mm
FoV phase	100.0 %
Slice thickness	7.0 mm
TR	8.6 ms
TE	4.00 ms
Averages	2
Concatenations	3
Filter	Normalize, Elliptical filter
Coil elements	1H

Contrast

TD	0 ms
MTC	Off
Magn. preparation	None
Flip angle	20 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution

Base resolution	256
-----------------	-----

Phase resolution	90 %
Phase partial Fourier	Off
Interpolation	On

PAT mode	None

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	Off
Normalize	On
Intensity	Medium
Cut off	20
Width	4
Unfiltered images	Off
B1 filter	Off
Raw filter	Off
Elliptical filter	On
Mode	Inplane

Geometry

Multi-slice mode	Sequential
Series	Interleaved

Saturation mode	Standard
Special sat.	None

Table position	H
Table position	0 mm
Inline Composing	Off

Tim CT mode	Off

System

1H	On
31P	Off

Positioning mode	REF
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Save uncombined	On
Coil Combine Mode	Sum of Squares
AutoAlign	---
Auto Coil Select	Default

Shim mode	Tune up
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
! Ref. amplitude 1H	200.000 V
Adjustment Tolerance	Auto
Adjust volume	
! Position	L1.2 A10.8 F21.5
! Orientation	C > T39.5
! Rotation	90.00 deg
! R >> L	192 mm
! F >> H	167 mm
! A >> P	191 mm

Physio

1st Signal/Mode	None
Segments	1

Tagging	None
Dark blood	Off

Resp. control	Off

Inline

Subtract	Off
Liver registration	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off

MapIt	None
Contrasts	1

Sequence

Introduction	On
Dimension	2D
Phase stabilisation	Off
Asymmetric echo	Allowed
Bandwidth	320 Hz/Px
Flow comp.	No

RF pulse type	Normal
Gradient mode	Normal
Excitation	Slice-sel.
RF spoiling	On

SIEMENS MAGNETOM Investigational_Device_7T syngo MR B17

\\USER\SHB\WPC-8922\SPA_31P\localizer

TA: 0:13 PAT: Off Voxel size: 1.1x1.0x7.0 mm Rel. SNR: 1.00 SIEMENS: gre

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	On
Start measurements	single

Routine

Slice group 1	
Slices	1
Dist. factor	20 %
Position	R1.8 A50.0 F13.4
Orientation	Sagittal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Slice group 2	
Slices	1
Dist. factor	20 %
Position	R1.8 A62.4 F20.0
Orientation	Transversal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Slice group 3	
Slices	1
Dist. factor	20 %
Position	R1.8 A64.9 F37.9
Orientation	Coronal
Phase enc. dir.	R >> L
Rotation	0.00 deg
Phase oversampling	0 %
FoV read	250 mm
FoV phase	100.0 %
Slice thickness	7.0 mm
TR	8.6 ms
TE	4.00 ms
Averages	2
Concatenations	3
Filter	Normalize, Elliptical filter
Coil elements	1H

Contrast

TD	0 ms
MTC	Off
Magn. preparation	None
Flip angle	20 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution

Base resolution	256
-----------------	-----

Phase resolution	90 %
Phase partial Fourier	Off
Interpolation	On

PAT mode	None

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	Off
Normalize	On
Intensity	Medium
Cut off	20
Width	4
Unfiltered images	Off
B1 filter	Off
Raw filter	Off
Elliptical filter	On
Mode	Inplane

Geometry

Multi-slice mode	Sequential
Series	Interleaved

Saturation mode	Standard
Special sat.	None

Table position	H
Table position	0 mm
Inline Composing	Off

Tim CT mode	Off

System

1H	On
31P	Off

Positioning mode	REF
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Save uncombined	On
Coil Combine Mode	Sum of Squares
AutoAlign	---
Auto Coil Select	Default

Shim mode	Tune up
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
! Ref. amplitude 1H	200.000 V
Adjustment Tolerance	Auto
Adjust volume	
! Position	R2.3 A48.3 F29.6
! Orientation	C > T39.5
! Rotation	90.00 deg
! R >> L	192 mm
! F >> H	167 mm
! A >> P	191 mm

Physio

1st Signal/Mode	None
Segments	1

Tagging	None
Dark blood	Off

Resp. control	Off

Inline

Subtract	Off
Liver registration	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off

MapIt	None
Contrasts	1

Sequence

Introduction	On
Dimension	2D
Phase stabilisation	Off
Asymmetric echo	Allowed
Bandwidth	320 Hz/Px
Flow comp.	No

RF pulse type	Normal
Gradient mode	Normal
Excitation	Slice-sel.
RF spoiling	On

SIEMENS MAGNETOM Investigational_Device_7T syngo MR B17

\\USER\SHB\WPC-8922\SPA_31P\localizer_cs1

TA: 5.5 s PAT: Off Voxel size: 1.1x1.0x5.0 mm Rel. SNR: 1.00 SIEMENS: gre

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	On
Start measurements	single

Routine

Slice group 1	
Slices	1
Dist. factor	20 %
Position	R1.8 A50.0 F13.4
Orientation	Sagittal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Phase oversampling	0 %
FoV read	250 mm
FoV phase	100.0 %
Slice thickness	5.0 mm
TR	8.6 ms
TE	4.00 ms
Averages	2
Concatenations	1
Filter	Normalize, Elliptical filter
Coil elements	1H

Contrast

MTC	Off
Magn. preparation	None
Flip angle	20 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution

Base resolution	256
Phase resolution	90 %
Phase partial Fourier	Off
Interpolation	On

PAT mode	None

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	Off
Normalize	On
Intensity	Medium
Cut off	20
Width	4
Unfiltered images	Off
B1 filter	Off
Raw filter	Off

Elliptical filter On
Mode Inplane

Geometry

Multi-slice mode	Sequential
Series	Interleaved

Saturation mode	Standard
Special sat.	None

Table position	H
Table position	0 mm
Inline Composing	Off

Tim CT mode	Off

System

1H	On
31P	Off

Positioning mode	REF
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Save uncombined	On
Coil Combine Mode	Sum of Squares
AutoAlign	---
Auto Coil Select	Default

Shim mode	Tune up
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
! Ref. amplitude 1H	200.000 V
Adjustment Tolerance	Auto
Adjust volume	
! Position	R2.3 A48.3 F29.6
! Orientation	C > T39.5
! Rotation	90.00 deg
! R >> L	192 mm
! F >> H	167 mm
! A >> P	191 mm

Physio

1st Signal/Mode	None
Segments	1

Tagging	None
Dark blood	Off

Resp. control	Off

Inline

Subtract	Off
Liver registration	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Wash - In	Off
Wash - Out	Off
TTP	Off

PEI	Off
MIP - time	Off

Maplt	None
Contrasts	1

Sequence

Introduction	On
Dimension	2D
Phase stabilisation	Off
Asymmetric echo	Allowed
Bandwidth	320 Hz/Px
Flow comp.	No

RF pulse type	Normal
Gradient mode	Normal
Excitation	Slice-sel.
RF spoiling	On

SIEMENS MAGNETOM Investigational_Device_7T syngo MR B17

\\USER\SHB\WPC-8922\SPA_31P\localizer_cs2

TA: 0:11 PAT: Off Voxel size: 1.0x1.0x3.0 mm Rel. SNR: 1.00 SIEMENS: gre

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	On
Start measurements	single

Routine

Slice group 1	
Slices	1
Dist. factor	20 %
Position	R1.8 A50.0 F13.4
Orientation	Sagittal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Phase oversampling	0 %
FoV read	250 mm
FoV phase	100.0 %
Slice thickness	3.0 mm
TR	6.0 ms
TE	2.60 ms
Averages	6
Concatenations	1
Filter	Normalize
Coil elements	1H

Contrast

MTC	Off
Magn. preparation	None
Flip angle	15 deg
Fat suppr.	None
Water suppr.	None
SWI	Off
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution

Base resolution	256
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	On
PAT mode	None
Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	Off
Normalize	On
Intensity	Medium
Cut off	20
Width	4
Unfiltered images	Off
B1 filter	Off
Raw filter	Off

Elliptical filter Off

Geometry

Multi-slice mode	Sequential
Series	Interleaved
Saturation mode	Standard
Special sat.	None
Table position	H
Table position	0 mm
Inline Composing	Off
Tim CT mode	Off

System

1H	On
31P	Off
Positioning mode	REF
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Save uncombined	On
Coil Combine Mode	Sum of Squares
AutoAlign	---
Auto Coil Select	Default
Shim mode	Tune up
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
! Ref. amplitude 1H	200.000 V
Adjustment Tolerance	Auto
Adjust volume	
! Position	R2.3 A48.3 F29.6
! Orientation	C > T39.5
! Rotation	90.00 deg
! R >> L	192 mm
! F >> H	167 mm
! A >> P	191 mm

Physio

1st Signal/Mode	None
Segments	1
Tagging	None
Dark blood	Off
Resp. control	Off

Inline

Subtract	Off
Liver registration	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On
Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off

MIP - time	Off

MapIt	None
Contrasts	1

Sequence

Introduction	On
Dimension	2D
Phase stabilisation	Off
Asymmetric echo	Off
Bandwidth	360 Hz/Px
Flow comp.	No

RF pulse type	Fast
Gradient mode	Normal
Excitation	Slice-sel.
RF spoiling	On

SIEMENS MAGNETOM Investigational_Device_7T syngo MR B17

\\USER\SHB\WPC-8922\SPA_31P\localizer_cs3

TA: 0:40 PAT: Off Voxel size: 1.0x1.0x2.0 mm Rel. SNR: 1.00 SIEMENS: gre

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	On
Start measurements	single

Routine

Slice group 1	
Slices	5
Dist. factor	10 %
Position	R1.8 A50.0 F13.4
Orientation	Sagittal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Phase oversampling	0 %
FoV read	250 mm
FoV phase	100.0 %
Slice thickness	2.0 mm
TR	6.0 ms
TE	3.00 ms
Averages	5
Concatenations	5
Filter	Normalize
Coil elements	1H

Contrast

TD	0 ms
MTC	Off
Magn. preparation	None
Flip angle	15 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution

Base resolution	256
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	On

PAT mode	None

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	Off
Normalize	On
Intensity	Medium
Cut off	20
Width	4
Unfiltered images	Off
B1 filter	Off

Raw filter	Off
Elliptical filter	Off

Geometry

Multi-slice mode	Sequential
Series	Interleaved

Saturation mode	Standard
Special sat.	None

Table position	H
Table position	0 mm
Inline Composing	Off

Tim CT mode	Off

System

1H	On
31P	Off

Positioning mode	REF
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Save uncombined	On
Coil Combine Mode	Sum of Squares
AutoAlign	---
Auto Coil Select	Default

Shim mode	Tune up
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
! Ref. amplitude 1H	200.000 V
Adjustment Tolerance	Auto
Adjust volume	
! Position	R2.3 A48.3 F29.6
! Orientation	C > T39.5
! Rotation	90.00 deg
! R >> L	192 mm
! F >> H	167 mm
! A >> P	191 mm

Physio

1st Signal/Mode	None
Segments	1

Tagging	None
Dark blood	Off

Resp. control	Off

Inline

Subtract	Off
Liver registration	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Wash - In	Off
Wash - Out	Off
TTP	Off

PEI	Off
MIP - time	Off

Maplt	None
Contrasts	1

Sequence

Introduction	On
Dimension	2D
Phase stabilisation	Off
Asymmetric echo	Off
Bandwidth	360 Hz/Px
Flow comp.	No

RF pulse type	Fast
Gradient mode	Normal
Excitation	Slice-sel.
RF spoiling	On

SIEMENS MAGNETOM Investigational_Device_7T syngo MR B17

\\USER\SHB\WPC-8922\SPA_31P\anat-T1w_acq-long

TA: 6:56 PAT: Off Voxel size: 1.0x1.0x1.0 mm Rel. SNR: 1.00 SIEMENS: tfl

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	On
Start measurements	single

Routine

Slab group 1	
Slabs	1
Dist. factor	50 %
Position	L0.2 A11.7 H3.2
Orientation	Transversal
Phase enc. dir.	R >> L
Rotation	90.00 deg
Phase oversampling	0 %
Slice oversampling	40.0 %
Slices per slab	160
FoV read	256 mm
FoV phase	71.9 %
Slice thickness	1.00 mm
TR	3000 ms
TE	1.87 ms
Averages	1
Concatenations	1
Filter	B1 filter
Coil elements	1H

Contrast

Magn. preparation	Slice-sel. IR
T1	1200 ms
Flip angle	8 deg
Fat suppr.	None
Water suppr.	None
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Off

Resolution

Base resolution	256
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	6/8
Slice partial Fourier	6/8
Interpolation	Off
PAT mode	None
Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	Off
Normalize	Off
B1 filter	On
Intensity	Medium
Unfiltered images	On

Raw filter	Off
Elliptical filter	Off

Geometry

Multi-slice mode	Single shot
Series	Interleaved
Table position	H
Table position	0 mm
Inline Composing	Off

System

1H	On
31P	Off
Positioning mode	FIX
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Save uncombined	Off
Coil Combine Mode	Sum of Squares
AutoAlign	---
Auto Coil Select	Default
Shim mode	Standard
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
! Ref. amplitude 1H	150.000 V
Adjustment Tolerance	Auto
Adjust volume	
! Position	R1.5 A11.1 F11.9
! Orientation	Transversal
! Rotation	-90.00 deg
! A >> P	169 mm
! R >> L	122 mm
! F >> H	120 mm

Physio

1st Signal/Mode	None
Dark blood	Off
Resp. control	Off

Inline

Subtract	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Sequence

Introduction	On
Dimension	3D
Elliptical scanning	Off
Asymmetric echo	Allowed
Bandwidth	390 Hz/Px
Flow comp.	No
Echo spacing	4.5 ms
RF pulse type	Fast

SIEMENS MAGNETOM Investigational_Device_7T syngo MR B17

Gradient mode
Excitation
RF spoiling

Normal
Slab-sel.
On

Table of contents

\\USER				
	SHB			
		WPC-8922		
			SPA_31P	
				anat-scout_ses-01p31
				anat-scout_ses-01p31
				anat-scout_ses-01p31
				fastestmap
				anat-T1w_acq-raw
				mrs-mrsi_acq-31Pv160
				mrs-mrsi_acq-31P
				anat-scout_acq-LD
				anat-scout_acq-shift
				___ CS _____
				localizer
				localizer_cs1
				localizer_cs2
				localizer_cs3
				----optional-----
				anat-T1w_acq-long